SARS-CoV-2 (COVID-19) Antigen Rapid Test
(Saliva)
Package Insert
For Self-testing
REF ICOV-803H English

INTENDED USE

The SARS-CoV-2 (COVID-19) Antigen Rapid Test (Saliva) is a single-use test kit intended to detect the novel coronavirus SARS-CoV-2 that causes COVID-19 in human saliva. This test is designed for home use¹ with self-collected saliva samples from symptomatic individuals who are suspected of being infected with SARS-CoV-2.

The SARS-CoV-2 (COVID-19) Antigen Rapid Test (Saliva) obtains a preliminary result only.

[SUMMARY]

This test device belongs to the B category. COVID-19 is an acute respiratory infectious disease. People are generally susceptible. Currently, the patients infected by the novel coronavirus are the main source of infection; asymptomatic infected people can also be an infectious source. Based on the current epidemiological investigation, the incubation period is 1 to 14 days, mostly 3 to 7 days. The main manifestations include fever, fatigue and dry cough. Nasal congestion, runny nose, sore throat, myalgia and diarrhea are found in a few cases.

[PRINCIPLE] The SARS-CoV-2 (COVID-19) Antigen Rapid Test (Saliva) is a qualitative membrane-based immunoassay for the detection of SARS-CoV-2 Antigens in human saliva specimen.

[REAGENTS]

The test device contains anti-SARS-CoV-2 antibodies.

[WARRANTY]

1. Read the entire package insert prior to performing test.

2. For self-testing in vitro diagnostic use only.

3. The test is for one time use only, do not reuse the test. Do not use after expiration date.

4. Do not eat, drink or smoke in the area where the specimens or kits are handled.

5. Do not touch the test window of the test device.

6. Do not use test if pouch is damaged.

7. Wash hands thoroughly before and after handling.

8. If the result is preliminary positive, carefully follow your local COVID guidelines/requirements.

9. The test should be used with an adult.

10. The used test should be discarded according to local regulations.

[STORAGE]

Store the test at 35-68°F (2-30°C). Do not open the pouch until ready for use. DO NOT FREEZE.

[ITEMS PROVIDED]

*Test Device *Package Insert

[ITEMS NOT PROVIDED]

*Timer

[TESTING]

Important: Do not place anything in the mouth including food, drink, gum or tobacco products for at least 10 minutes prior to collection. Wash your hands with soap and water for at least 20 seconds before testing. If soap and water are not available, use hand sanitizer with at least 60% alcohol.

Allow the test device to reach room temperature (15-30°C) prior to testing.

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