# ALL™ SARS-CoV-2 and Influenza A+B Antigen Combo Rapid Test (Nasal Swab)

REF ISIN-525H | English

A rapid test for the qualitative detection of SARS-CoV-2 Nucleocapsid Protein, Influenza A and Influenza B nucleoproteins antigens present in nasal swab specimens.

For self-testing in vitro diagnostic use.

## 【PROCEDURE】

Wash your hands with soap and water for at least 20 seconds before and after test. If soap and water are not available, use hand sanitizer with at least 60% alcohol.

Remove the cover of the tube with extraction buffer and place the tube in the tube holder in the box.

Nasal swab specimen collection

1. Remove the sterile swab from the pouch. Do not touch the soft tip of the swab.

2. Insert the swab into your nostril until you feel slight resistance (approx. 2cm up your nose). Slowly twist the swab, drawing it along the inside of your nasal cavity for 5-10 times against the nasal wall.

**Note:**

The nasal mucosa is damaged or bleeding, nasal swab collection is not recommended.

If you are still uncomfortable, please wear a face mask. When children, you may not need to insert the swab so far into the nostril. For very young children, you may need another person to steady the child's head while swabbing.

3. Gently remove the swab.

4. Using the same swab, repeat step 2 in your other nostril.

5. Withdraw the swab.

**Specimen preparation:**

1. Place the swab into the extraction tube, ensure it is touching the bottom and stir the swab to mix well. Press the swab head against to the tube and rotate the swab for 10-15 seconds.

2. Remove the tube by squeezing the swab head against the inside of the extraction tube.

3. Place the swab in a plastic bag.

4. Close the cap or fit the tube top onto the tube.

5. Insert the tube into the extraction tube.

6. Insert the tube into the tube holder in the box.

7. Insert the tube into the tube holder in the box.

8. Insert the tube into the tube holder in the box.

9. Insert the tube into the tube holder in the box.

10. Insert the tube into the tube holder in the box.

11. Insert the tube into the tube holder in the box.

12. Insert the tube into the tube holder in the box.

13. Insert the tube into the tube holder in the box.

14. Insert the tube into the tube holder in the box.

15. Insert the tube into the tube holder in the box.

16. Insert the tube into the tube holder in the box.

17. Insert the tube into the tube holder in the box.

18. Insert the tube into the tube holder in the box.

19. Insert the tube into the tube holder in the box.

20. Insert the tube into the tube holder in the box.

21. Insert the tube into the tube holder in the box.

22. Insert the tube into the tube holder in the box.

23. Insert the tube into the tube holder in the box.

24. Insert the tube into the tube holder in the box.

25. Insert the tube into the tube holder in the box.

26. Insert the tube into the tube holder in the box.

27. Insert the tube into the tube holder in the box.

28. Insert the tube into the tube holder in the box.

29. Insert the tube into the tube holder in the box.

30. Insert the tube into the tube holder in the box.

31. Insert the tube into the tube holder in the box.

32. Insert the tube into the tube holder in the box.

33. Insert the tube into the tube holder in the box.

34. Insert the tube into the tube holder in the box.

35. Insert the tube into the tube holder in the box.

36. Insert the tube into the tube holder in the box.

37. Insert the tube into the tube holder in the box.

38. Insert the tube into the tube holder in the box.

39. Insert the tube into the tube holder in the box.

40. Insert the tube into the tube holder in the box.

41. Insert the tube into the tube holder in the box.

42. Insert the tube into the tube holder in the box.

43. Insert the tube into the tube holder in the box.

44. Insert the tube into the tube holder in the box.

45. Insert the tube into the tube holder in the box.

46. Insert the tube into the tube holder in the box.

47. Insert the tube into the tube holder in the box.

48. Insert the tube into the tube holder in the box.

49. Insert the tube into the tube holder in the box.

50. Insert the tube into the tube holder in the box.

51. Insert the tube into the tube holder in the box.

52. Insert the tube into the tube holder in the box.

53. Insert the tube into the tube holder in the box.

54. Insert the tube into the tube holder in the box.

55. Insert the tube into the tube holder in the box.

56. Insert the tube into the tube holder in the box.

57. Insert the tube into the tube holder in the box.

58. Insert the tube into the tube holder in the box.

59. Insert the tube into the tube holder in the box.

60. Insert the tube into the tube holder in the box.

61. Insert the tube into the tube holder in the box.

62. Insert the tube into the tube holder in the box.

63. Insert the tube into the tube holder in the box.

64. Insert the tube into the tube holder in the box.

65. Insert the tube into the tube holder in the box.

66. Insert the tube into the tube holder in the box.

67. Insert the tube into the tube holder in the box.

68. Insert the tube into the tube holder in the box.

69. Insert the tube into the tube holder in the box.

70. Insert the tube into the tube holder in the box.

71. Insert the tube into the tube holder in the box.

72. Insert the tube into the tube holder in the box.

73. Insert the tube into the tube holder in the box.

74. Insert the tube into the tube holder in the box.

75. Insert the tube into the tube holder in the box.

76. Insert the tube into the tube holder in the box.

77. Insert the tube into the tube holder in the box.

78. Insert the tube into the tube holder in the box.

79. Insert the tube into the tube holder in the box.

80. Insert the tube into the tube holder in the box.

81. Insert the tube into the tube holder in the box.

82. Insert the tube into the tube holder in the box.

83. Insert the tube into the tube holder in the box.

84. Insert the tube into the tube holder in the box.

85. Insert the tube into the tube holder in the box.

86. Insert the tube into the tube holder in the box.

87. Insert the tube into the tube holder in the box.

88. Insert the tube into the tube holder in the box.

89. Insert the tube into the tube holder in the box.

90. Insert the tube into the tube holder in the box.

91. Insert the tube into the tube holder in the box.

92. Insert the tube into the tube holder in the box.

93. Insert the tube into the tube holder in the box.

94. Insert the tube into the tube holder in the box.

95. Insert the tube into the tube holder in the box.

96. Insert the tube into the tube holder in the box.

97. Insert the tube into the tube holder in the box.

98. Insert the tube into the tube holder in the box.

99. Insert the tube into the tube holder in the box.

100. Insert the tube into the tube holder in the box.

101. Insert the tube into the tube holder in the box.

102. Insert the tube into the tube holder in the box.

103. Insert the tube into the tube holder in the box.

104. Insert the tube into the tube holder in the box.

105. Insert the tube into the tube holder in the box.

106. Insert the tube into the tube holder in the box.

107. Insert the tube into the tube holder in the box.

108. Insert the tube into the tube holder in the box.

109. Insert the tube into the tube holder in the box.

110. Insert the tube into the tube holder in the box.

111. Insert the tube into the tube holder in the box.

112. Insert the tube into the tube holder in the box.

113. Insert the tube into the tube holder in the box.

114. Insert the tube into the tube holder in the box.

115. Insert the tube into the tube holder in the box.

116. Insert the tube into the tube holder in the box.

117. Insert the tube into the tube holder in the box.

118. Insert the tube into the tube holder in the box.

119. Insert the tube into the tube holder in the box.

120. Insert the tube into the tube holder in the box.

121. Insert the tube into the tube holder in the box.

122. Insert the tube into the tube holder in the box.

123. Insert the tube into the tube holder in the box.

124. Insert the tube into the tube holder in the box.

125. Insert the tube into the tube holder in the box.

126. Insert the tube into the tube holder in the box.

127. Insert the tube into the tube holder in the box.

128. Insert the tube into the tube holder in the box.

129. Insert the tube into the tube holder in the box.

130. Insert the tube into the tube holder in the box.

131. Insert the tube into the tube holder in the box.

132. Insert the tube into the tube holder in the box.

133. Insert the tube into the tube holder in the box.

134. Insert the tube into the tube holder in the box.

135. Insert the tube into the tube holder in the box.

136. Insert the tube into the tube holder in the box.

137. Insert the tube into the tube holder in the box.

138. Insert the tube into the tube holder in the box.

139. Insert the tube into the tube holder in the box.

140. Insert the tube into the tube holder in the box.

141. Insert the tube into the tube holder in the box.

142. Insert the tube into the tube holder in the box.

143. Insert the tube into the tube holder in the box.

144. Insert the tube into the tube holder in the box.

145. Insert the tube into the tube holder in the box.

146. Insert the tube into the tube holder in the box.

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